

Manual Instruction

1. Connect physiotherapy electrode with lead wire.
2. Tear off the protective film before use.
3. Clean the area that needs treating.
4. Attach the self-adhesive electrode to the part or the corresponding acupressure to be treated and compress it.
5. After treatment, separate the electrode and the wire, then cover the protective film.

Caution

- ◇ Never use at the same time as the following kinds of medical electrical devices. Doing so may cause an accident or the medical device to malfunction and result in significant bodily harm.
 - ◇ Implanted medical devices, such as a pacemaker
 - ◇ Medical electrical equipment used to sustain life, such as heart-lung machines.
 - ◇ Wearable medical devices, such as EKGs
- The following persons should not use this product. Doing so may cause an accident or problems to the body or skin.
- ◇ Persons with heart disease
 - ◇ Women who are pregnant or have just given birth
- Anyone with the following conditions should consult a physician prior to using this product. Not doing so may cause an accident or problems to the body or skin.
- ◇ Persons prone to allergies
 - ◇ Persons with an impairment in perception or problems with their skin (infection, inflamed skin or scratches)
 - ◇ Persons with blockages of blood flow
 - ◇ Persons taking medications
 - ◇ Persons who feel anything unusual with their cranial nerves (epilepsy, etc.)
 - ◇ Persons who sense abnormalities to their heart or cranial nerves
 - ◇ Persons with an impairment in perception due to a significant peripheral circulatory disorder, such as from diabetes
 - ◇ Persons with a fever (38 °C or above)
 - ◇ Persons who need rest
 - ◇ Persons with an acute (painful) condition, such as spine fracture, sprain, pulled muscle, etc.
 - ◇ Persons with a bone disorder, such as osteoporosis
 - ◇ Persons with abnormal blood pressure
 - ◇ Persons experiencing any specific physical abnormalities
 - ◇ Persons receiving treatment from a medical institution
-